







# DR400 Functions and Modes

## DR400 FUNCTIONS:

FUNCTION	DO THIS	LIGHT RESPONSE:	MORE INFORMATION
TO START RECORDING	1. Remove DR400 from docking station and allow to go into Standby mode AND 2. Hold the PRESS button down until green light goes out (about 3 seconds)	DR400 will flash for 60 minutes  → ○	The DR400 flashes for 60 minutes to tell you that recording has started (30 seconds for DR400 v5.04)
		DR400 shows solid orange light 	Recording did not start as there is either old data OR no Patient ID on the DR400. Also, you may see if the recording has stopped at the end of "Days"
		DR400 does not respond ○	Recording did not start. Most likely, the DR400 battery is not charged. If charged or plugged into docking station and no light is visible, contact Technical Support for further assistance
TO RECORD EVENT	While recording, quickly press the PRESS button	DR400 flashes green slowly for 90 seconds 	An event is being saved.
TO VIEW ECG	While recording and with PCPatch running, hold PRESS button down until DR400 flashes green	DR400 will flash green rapidly when transmitting Bluetooth data  → ○	Takes a few seconds to open window. The DR400 must be paired with the NorthEast Bluetooth USB and the PCPatch utility must be running on PC where the Bluetooth USB is attached.
TO STOP RECORDING, SAVE ECG & UPDATE SETTINGS	Put DR400 into docking station that is attached to PC where PCPatch utility is installed	DR400 starts charging and is ready for PC Patch n/a	Use PCPatch to Preview, Copy and/or erase the flash.dat file. Also use PC Patch to update Settings, and pair DR400 with Bluetooth USB and/or Gateway (for Event).



## DR400 Modes

When plugged into docking station, the DR400 light has only

**two modes:**


1. Charging if the light flashes orange, and
2. Fully Charged when flashing green.

When not plugged in, the DR400 is either **Recording or in Standby mode.**

**Not sure if the DR400 is Recording or in Standby? Quickly press the PRESS button and observe:**

1. If the DR400 flashes green for about 30 seconds, it is **Recording**.
2. If light is green, but no flashing occurs, the DR400 is in **Standby** and ready for next patient.
3. If the light is orange, DR400 is not recording and cannot be started as is. Before next use, use PCPatch to Preview, Save, Erase and enter new patient ID. Recorder may need to be charged.
4. If no lights, the DR400 battery needs to be charged.

Rxonly

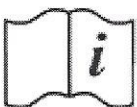

www.nemont.com



MediMark Europe SARL  
11 Rue Emile Zola  
38100 Grenoble France

