




Holter Hook-Up Chart

CANINE - CR160

Our model is Ch. Encore's Stetson

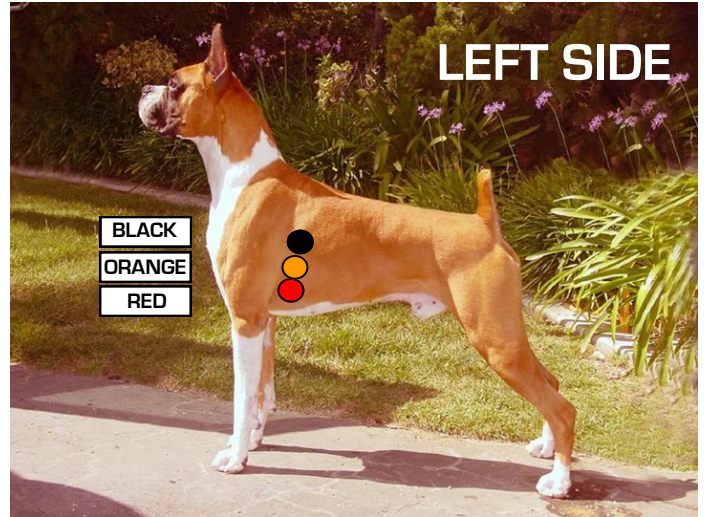

AFTER CONNECTING THE DOG

1. Open lid of Holter.
2. Open head-bar with thumb release.
3. Insert Cassette with label side up. (Full tape reel should be on the left side).
4. Insert new 9V battery. Polarity (+ / -) doesn't matter.
5. Close head-bar. Make sure it "clicks" twice.
6. Visually check that tape is turning. (It is very slow).
7. Write animal name and start time on cassette.
8. Close lid and secure Holter Monitor.

AFTER 24 HOURS OF RECORDING

1. Remove and discard battery from Holter.
2. Remove cassette from Holter.
3. Complete ALBA Medical Holter Monitoring Request Form.
This form is on the Animal Health section of www.albamedical.com.
4. Send cassette, form, diary and \$30 payment to ALBA Medical for scanning.