



TO CONNECT THE PATIENT

More information on back

1. Fully charge the DR400 using the USB Docking Station or AC Adaptor. The DR400 will flash orange while charging and flash green every 4 seconds when fully charged.
2. Start the PC Patch program on the PC.
3. Attach the DR400 to the Docking Station.
3. On the PC Patch program click "Erase DR400". (Warning: This will erase previous patient).
4. Click "New Patient". Enter a patient ID (for example the patient's name).
5. Click "Update Recorder and Close".
6. Disconnect the DR400 from the Docking Station.
7. Connect three electrodes to the DR400. Spread out the three electrodes to give the biggest spread as possible.
8. Remove the backing from the electrodes and connect to the dog as in above hook-up chart.
******* PLEASE NOTE:** The machine must be positioned upside down (see picture on right).
9. Press and hold the PRESS button on the DR400 until the green light goes out. The DR400 will flash green for up to 60 minutes to show you that it is recording.
10. If the Bluetooth dongle is installed, and the patient is within 30 feet of the computer, you can see the ECG appear in the PC Patch program. Press and hold the PRESS button until you see a quick flashing of the green light. (After this is done, the green light will stop blinking; this is normal).